


# USE OF THE GENUAIR®



Drug : \_\_\_\_\_  
 Indication (maintenance or rescue or maintenance+rescue): \_\_\_\_\_  
 Number of inhalations: \_\_\_\_\_ Number of times per day: \_\_\_\_\_  
 Other information: \_\_\_\_\_

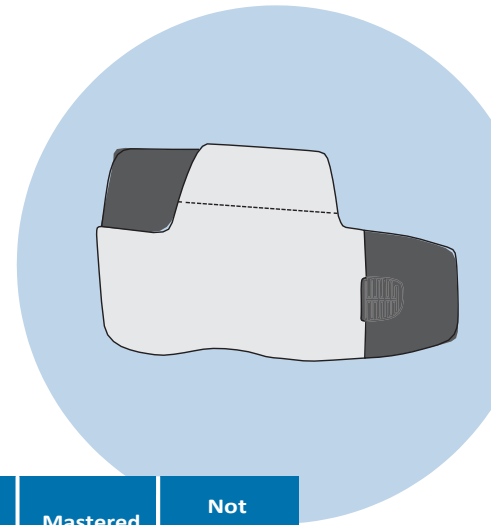

		Mastered	Not mastered
1	Prepare the device: a. Remove the protective cap by pressing lightly on the arrows on each side of the cap		
2	Hold the inhaler horizontally and press the button all the way down and release: a. A click will be heard b. The window turns green indicating that the dose is ready		
3	Breathe out normally next to the device, emptying the lungs without forcing		
4	Place the mouthpiece between the lips and teeth		
5	Breathe in through the mouth strongly and deeply: a. A click is heard to indicate that the inhaler is being used properly: keep breathing in to take the full dose b. We can suggest to breathe in "quickly" if the patient is unable to trigger the device c. The window will turn red again to indicate that the dose has been taken		
6	Move the device away from the mouth		
7	Respect a pause: hold your breath for 5 to 10 seconds		
8	Breathe out through the nose		
9	Replace the protective cap		

## REMARKS

To limit the adverse effects of the medication, rinse your mouth with water (do not swallow the water).

**MAINTENANCE:** \_\_\_\_\_

**HOW TO KNOW IF IT IS EMPTY:** \_\_\_\_\_



**RQESR**

RÉSEAU QUÉBÉCOIS D'ÉDUCATION  
EN SANTÉ RESPIRATOIRE

La référence des professionnels