


# USE OF THE RESPIMAT \*<sup>®</sup>

\* Daily use

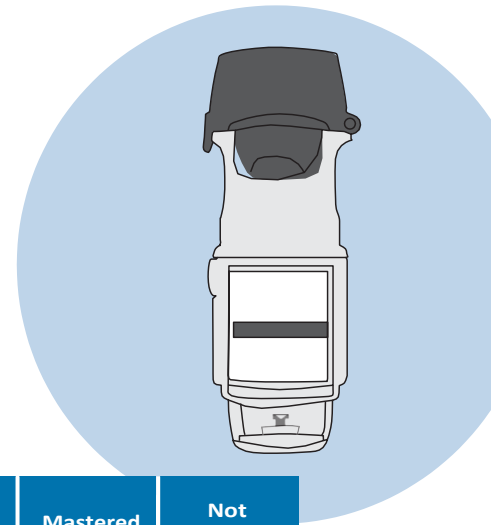

Drug : \_\_\_\_\_  
 Indication (maintenance or rescue or maintenance+rescue): \_\_\_\_\_  
 Number of inhalations: \_\_\_\_\_ Number of times per day: \_\_\_\_\_  
 Other information: \_\_\_\_\_



		Mastered	Not mastered
1	Prepare the device by holding it vertically with the cap closed		
2	Turn the base half a turn in the direction of the arrows until it clicks		
3	Open the cap completely		
4	Breathe out normally next to the device, emptying the lungs without forcing		
5	Place the mouthpiece horizontally between the lips and teeth: a. Do not block the air intake		
6	Start breathing in through the mouth slowly and deeply and press the activation button once to release the dose of medication		
7	Continue to breathe in slowly and deeply until you have completed the breath		
8	Move the device away from the mouth		
9	Respect a pause: hold your breath for 5 to 10 seconds		
10	Breathe out through the nose		
11	Close the cap		
12	Repeat steps 1-11 for each prescribed inhalation		

## NOTE

For the first use, if possible, have the cartridge inserted into the device by the pharmacy

## REMARKS

To limit the adverse effects of the medication, rinse your mouth with water (do not swallow the water).

**ACTIVATION / REACTIVATION :**

**MAINTENANCE:**

**HOW TO KNOW IF IT IS EMPTY:**



**RQESR**

RESEAU QUÉBÉCOIS D'ÉDUCATION  
EN SANTÉ RESPIRATOIRE  
La référence des professionnels